



PATENTED

Ergo Anti-Tipper

Quick-Release to Overcome Curbs in ONE Single Step




Curb-Climbing Mode


Step on the tipping-lever, quick-release the anti-tipper, and tip over wheelchair in one single step.

Safety Mode

Push down the tipper to ensure wheelchair stability.

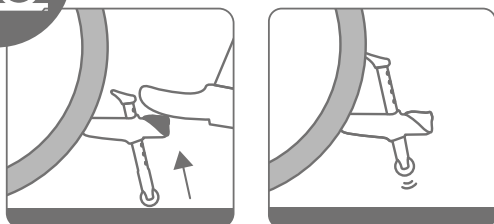

Before climbing curb, step on the tipping lever to quick-release the anti-tipper.

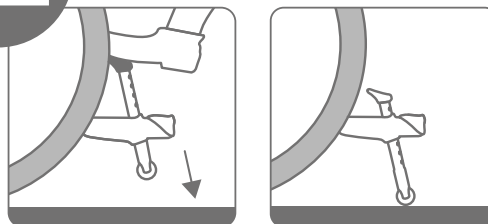

After climbing a curb, make sure you push down the tipper to prevent wheelchair from tipping over.

Standard For Models

Flexx Adapt

Optional For Models

Ergo 2
Ergo 3
Ergo Lite
Ergo Lite 2



Scan to learn more about Ergo Anti-tipper



Why do you need the Ergo Anti-tipper?



How to Install the Ergo Anti-tipper

KARMA MEDICAL PRODUCTS CO., LTD.

T +886-5-206 6688

F +886-5-206 7788

Email : globalsales@karma.com.tw

karma

www.KarmaMedical.com